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NEBULIZER Aerosol Therapy System

INSTRUCTION MANUAL

Thank you for choosing our Nebulizer Gamma™ designed to carry out aerosol therapy of acute and chronic diseases of upper and lower respiratory passages. This device allows for easy and effective transportation of big medicine doses directly to the disease centre.

We are sure that having appraised worthily the high quality and reliability of this device you will become a regular user of the products of Gamma Trade Mark.

Before starting to use the nebulizer system please study the instruction manual carefully. The instruction manual offers all information you need to correctly carry out aerosol therapy.

ATTENTION


Please, use only those medicines that were prescribed to you by your doctor, DO NOT execute self-treatment and DO NOT change the methods prescribed without consulting your doctor.

CONTENT

1. COMPONENTS AND DESCRIPTION OF YOUR NEBULIZER SYSTEM FAMILY
2. GETTING READY FOR USAGE
 - 2.1. ASSEMBLING THE DEVICE
 - 2.2. REPLACEMENT OF NEBULIZER
 - 2.3. REPLACEMENT OF FILTER
3. CARRYING OUT NEBULIZING THERAPY
4. CLEANING AND DISINFECTION
5. POSSIBLE PROBLEMS AND HOW TO SOLVE THEM
6. TECHNICAL SPECIFICATIONS
7. WARRANTY
8. WHAT IS NEBULIZING THERAPY

1. COMPONENTS AND DESCRIPTION OF YOUR NEBULIZER SYSTEM FAMILY

Your nebulizer system includes compressor housing, power cord, air tube, angled mouthpiece, nosepiece, MEDELJET BASIC nebulizer, 5 spare filters, adult mask, Instruction Book, guarantee card, carrying case (only for SILVER model), packing.

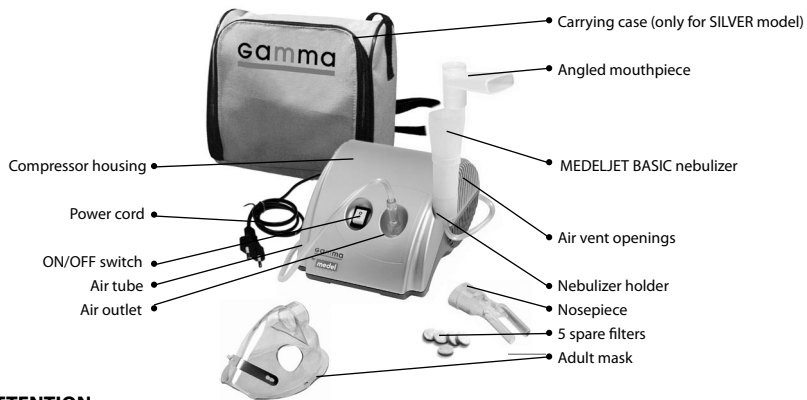ATTENTION:

- Do not leave the unit plugged when not in use: unplug the device from the wall socket when not operating.
- Follow the manufacturer's instructions for installing the device. The manufacturer cannot be held responsible for any damage caused to persons, animals or things by incorrect installation.
- In case the power supply cord becomes damaged, address to the technical servicing for replacement; the address is given in your guarantee card.
- The power supply cord should always be fully unwound in order to prevent dangerous overheating.

- Before performing any maintenance or cleaning operation, disconnect the device from the main supply, by unplugging it or by turning the main supply switch off.
- Some parts of the unit are so small that they could be swallowed by children. Do not let unattended children have access to the unit.
- If you decide not to use the device any longer, it is recommended to dispose of it according to the regulations in force.
- Use the "nose piece" accessory only if expressly indicated by your doctor and paying attention NEVER you introduce inside the nose the nasal bifurcation, but only bring it as close as possible.
- Make the treatment only using the accessory recommended by your doctor for the therapy.
- Verify that the medications prescribed by your doctor can be used with common compressor nebulizer systems for aerosol therapy.
- Never use essential oils since they corrode the plastic - made pieces.

2. GETTING READY FOR USAGE

2.1. ASSEMBLING THE DEVICE

1. As soon as you have unpacked the device, check it for visible damages or defects. In particular, check for cracks in the plastic housing, which could expose electrical components.

2. Check for accessories integrity.

3. Before using the device, proceed with the cleaning operations as described in the "CLEANING AND DISINFECTION" section.

4. Open the nebulizer (see photo) by turning the top in an anticlockwise direction.

5. Make sure the water conduction cone is properly fitted onto the air conduction cone inside the nebulizer (see photo).


6. Put the prescribed quantity of medication into the nebulizer (see photo).

7. Close the nebulizer by turning the two parts clockwise, paying attention that they are thoroughly sealed.

8. Connect one end of the air tube to the nebulizer (see pic., section 1), and the other end to the air outlet on the device (see the photo).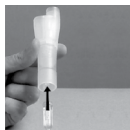

9. Connect the angled mouthpiece onto the nebulizer (see the pic., section 1 and photo).

10. If you need to use the nosepiece or the mask (see the pic., section 1), take the angled mouthpiece off and connect the nosepiece or the mask directly onto the nebulizer (see photo).

11. Plug the device into the wall socket, making sure the electrical rating shown on the rating plate on the bottom of the unit corresponds to your main voltage.

ATTENTION:

Follow the manufacturer's instructions for installing the device. The manufacturer cannot be held responsible for any damage caused to persons, animals or things by incorrect installation.

2.2. REPLACEMENT OF NEBULIZER


The nebulizer must be replaced after a long period of inactivity, or when it is obstructed by dry medication, dust, etc.

Your nebulizer should last from 6 months to 1 year with general use.
Use original MEDELJET nebulizers only.

2.3. REPLACEMENT OF THE FILTER

The filter must be replaced approximately after 30 hours of use, or when it turns dirty; open the filter holder to replace the filter (see photo) with a new one. Use original filters only.

Do not use the device without filter.

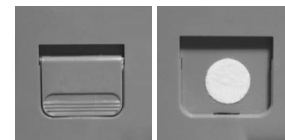3. CARRYING OUT NEBULIZING THERAPY

Note

Remove the compressor from carrying case before use (only for SILVER model).

1. To start the treatment, switch the device on by pressing the ON/OFF switch onto the "I" position.

ATTENTION:

This device is for intermittent use: 30 min. ON/30 min. OFF.

2. Inhale the aerosol solution using the prescribed accessory in calm way. You should sit at the table with one's back upright, relaxed; otherwise you will constrict your respiratory passages and impair the effect of inhalation..

3. When the treatment has been completed, switch the unit off by setting the switch position "O" and disconnect the plug from the wall socket.

4. Clean the nebulizer and accessories as described in the "CLEANING AND DISINFECTION" section.

ATTENTION

- Only use the device for its intended purpose as described in this instruction manual, therefore as a compressor nebulizer system for aerosoltherapy, following the indications of your doctor. Any other form of use constitutes an improper use and is therefore dangerous. The manufacturer cannot be held liable for any damage caused by improper, incorrect or unreasonable use, or if the unit is connected to electrical installations which do not comply with safety regulations.
- Keep this manual for future reference.
- Do not operate the unit in the presence of any anaesthetic, inflammable mixtures or oxygen.
- The correct device operating can be affected by electromagnetic interference which exceeds the limits indicated by the European standards in force.
- In case of failure and/or malfunction, read the "POSSIBLE PROBLEMS AND HOW TO SOLVE THEM" section. Do not handle nor open the compressor housing.
- When using any electrical appliance certain important safety measures must always be observed, including the following:
 - use only original accessories and components;
 - never submerge the unit in water;
 - never touch the unit when your hands are wet or moist;
 - do not leave the unit outdoor;
 - the unit, when operating, must be placed on a stable and horizontal surface;
 - make sure the air vent openings are not obstructed;
 - do not allow unsupervised children or infirm persons to use the unit;
 - do not disconnect the unit by simply pulling out the plug from the wall socket.
- If the power plug provided with the device does not fit your wall socket, have the plug changed by an electrical appliance dealer. Do not use any adapter, simple or multiple, and/or extension cable. In case their use is necessary, make sure they are in compliance with safety standards, paying attention they do not exceed the maximum load.
- The power supply cord cannot be replaced by the user. In case the power supply cord becomes damaged, address to the technical servicing authorized by the manufacturer for replacement.

4. CLEANING AND DISINFECTION.

Use a clean dry cloth and mild detergents for cleaning the unit.

ATTENTION:

During cleaning operations, make sure the internal parts of the unit are not in contact with liquids and the power cord is not connected.

After each treatment:

The nebulizer, angled mouthpiece and nosepiece must be disinfected by boiling with water (max. 10 minutes) or in autoclave (max. 121°C). We suggest you disinfect the mask using cold disinfecting liquids (solutions with hypochlorite). Dry the nebulizer using a clean cloth; after connect it to the air outlet, switch the device on and let it work freely for 10-15 minutes.

Do not boil nor autoclave the air tube and the mask.

Following carefully the cleaning and disinfecting instructions is very important for the product performance and for a successful therapy.

5. POSSIBLE PROBLEMS AND HOW TO SOLVE THEM

Problem	Troubleshooting
The device does not switch on	<ol style="list-style-type: none">1. Make sure the plug is firmly fitted to the wall socket;2. Make sure that the device has been operating within operating limits indicated in this instructions manual (30 min. ON/30 min. OFF).
The device does not nebulize, or it nebulizes very weakly	<ol style="list-style-type: none">1. Make sure that the ends of the air tube are fitted tightly onto the main unit and onto the nebulizer;2. Check whether the nebulizer is empty from medication or full of an excessive medication quantity (MAX. 6ml);3. Check whether the nebulizer nozzle is obstructed;4. Check whether the air conduction cone is inserted (pic. C).

Should the device not start working properly again, please contact the Service Centre; the address is stated in the guarantee card.

6. TECHNICAL SPECIFICATIONS

Model	FAMILY
Electricity supply	see rating label on the device bottom
Nebulization rate	0,28 ml/min
Particle size	MMAD 1,9 µm (NaCl 0,9%) - API AEROSIZER MMD 4,99 µm - INSITEC MALVERN
Compressor max. pressure	230 kPa (2,30 bar)
Weight	1,55 kg
Size	(H) 167 x (L) 145 x (P) 97 mm
Noise Level	55 dBA
Class II Device	Double Safety Insulation
Type B Device	(device with specific protection against electrical hazards)
Reference to Standards	Electric Safety Standards CEI EN 60601-1 Electromagnetic Compatibility according to CEI EN 60601-1-2 Class IIa Medical Device according to 93/42/EEC "Medical Device" Directive
Storage Condition	Temperature: MIN -25°C-MAX 70°C Humidity: MIN 10%RH-MAX 95%RH Atmospheric Pressure: 700hPa-1060hPa
Operating Conditions	Temperature: MIN 10°C-MAX 40°C Humidity: MIN 10%RH-MAX 95%RH Atmospheric Pressure: 700hPa-1060hPa

Technical modifications reserved!

DEVICE SYMBOLS

I	ON
○	OFF
⚡	Type B
⚠	Read the instructions
~	Alternating Current
□	Class II

CE 0123

This device fulfils the provisions of 93/42/EEC Directive.

7. WARRANTY

The compressor nebulizer system device is covered by a 3-year warranty from date of purchase for any defects in the design or in the materials used. The warranty does not cover the accessories supplied with the device and those parts subject to normal wear and tear.

The warranty does not apply to damage caused by improper handling, accidents, not following the operating instructions or self-maintained alterations made to the device.

The manufacturer shall not be liable for any damages to people, property or pets, either directly or indirectly, caused by the non-compliance with the instruction book.

The warrantee is only valid upon presentation of the warrantee card which was correctly filled in and sealed.

8. WHAT IS NEBULIZING THERAPY

What is a nebulizer?

The name of the device comes from the Latin word nebula that means fog, cloud. The nebulizer system is a device that turns a liquid drug into a fine aerosol under the influence of the compressed air. Aerosol is a drug suspension. So, nebulizing therapy is an inhalation therapy with the help of aerosols.

Inhalation therapy has been used for treatment of lung diseases for a long time. The ancient civilizations of Egypt, India, China, the Middle East, Greece are known to have used inhalations of evaporations from menthol, eucalyptus, different plants.

For patients with lung diseases nebulizing therapy seems to be the most rational one since big drug doses come very fast to the respiratory tract directly.

How does the nebulizer system work?

Nebulizer directs an aerosol containing larger and smaller particles to the respiratory tract. The gravity makes the larger particles accumulate on the walls of the upper respiratory passages while the smaller particles have enough time to reach the lower bronchial branches. Therefore the drug comes to all parts of the patient's bronchial tree, including the smallest bronchi and alveole, and gets into the blood vessels on the walls of the alveole in big therapeutic doses with tiny or none side effects. Some drugs are very slow to be absorbed from the alimentary canal or are affected dramatically by the first passage through the liver. In such cases the way out is to use inhalation therapy. Nebulizing therapy can be used by patients of any age, it's suitable for small children and infirms.

What is nebulizing therapy used for?

Nebulizer systems are used by the patients with:

- acute respiratory diseases;
- obstructive pulmonary disease;
- chronic inflammatory processes in respiratory passages such as chronic bronchitis, bronchial asthma (incl. chronic or unstable ones, exacerbation of asthma), chronic pharyngitis;
- respiratory passages infections;
- abnormalities in secretion quality and secreting in bronchi;
- different types of coughing;

Nebulizing therapy helps to decrease the level of inflammation activity, to improve respiratory passage drain function, to lessen the bronchismus, to make the edema smaller and to protect the mucous membrane, etc.

What are the advantages of nebulizing therapy compared to other means of drug delivery?

Nebulizing therapy gives a possibility to quickly effect the inflammatory zone by big drug doses which increases the influence of the treatment and reduces the risk of side effects to other organs. Nebulizing therapy is the easiest way of inhalation treatment and it is also an economical means of treatments from the point of view of purposeful use of drug. Aerosol treatment with the help of nebulizing systems can be used for children, elderly and infirm patients.

Please, use only those drugs that were prescribed to you by your doctor, DO NOT execute self-treatment and DO NOT change the methods/frequency prescribed without consulting your doctor.

CE 0123

Manufacturer (93/42/EEC):

Designed, engineered and imported in E.U. by:

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MEDEL S.p.A.