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ULTRASONIC NEBULIZER Aerosol Therapy System

INSTRUCTION MANUAL

Thank you for choosing our Nebulizer Gamma® designed to carry out aerosol therapy of acute and chronic diseases of upper and lower respiratory passages. This device allows for easy and effective transportation of big medicine doses directly to the disease centre.

We are sure that having appraised worthily the high quality and reliability of this device you will become a regular user of the products of Gamma Trade Mark.

Before starting to use the nebulizer system please study the instruction manual carefully. The instruction manual offers all information you need to correctly carry out aerosol therapy.

ATTENTION

Please, use only those medicines that were prescribed to you by your doctor, DO NOT execute self-treatment and DO NOT change the methods prescribed without consulting your doctor.


CONTENT

1. COMPONENTS AND DESCRIPTION OF YOUR NEBULIZER SYSTEM MODEL COMPACT
2. GETTING READY FOR USAGE
 - 2.1. ASSEMBLING THE DEVICE
 - 2.2. REPLACEMENT OF FILTER
 - 2.3. USING CAR LIGHTER ADAPTER (OPTIONAL)
 - 2.4. USING THE RECHARGEABLE BATTERY PACK
3. CARRYING OUT AEROSOL THERAPY
4. CLEANING AND DISINFECTION
5. POSSIBLE PROBLEMS AND HOW TO SOLVE THEM
6. TECHNICAL SPECIFICATIONS
7. WARRANTY
8. WHAT IS NEBULIZING THERAPY

1. COMPONENTS AND DESCRIPTION OF YOUR NEBULIZER MODEL COMPACT

Your nebulizer system set includes main body, adapter, 2 air filters, adult mask, pediatric mask, mouth piece, nasal piece, AC adapter, Instruction Book, guarantee card, packing.

The picture shows nebulizer model **Compact**

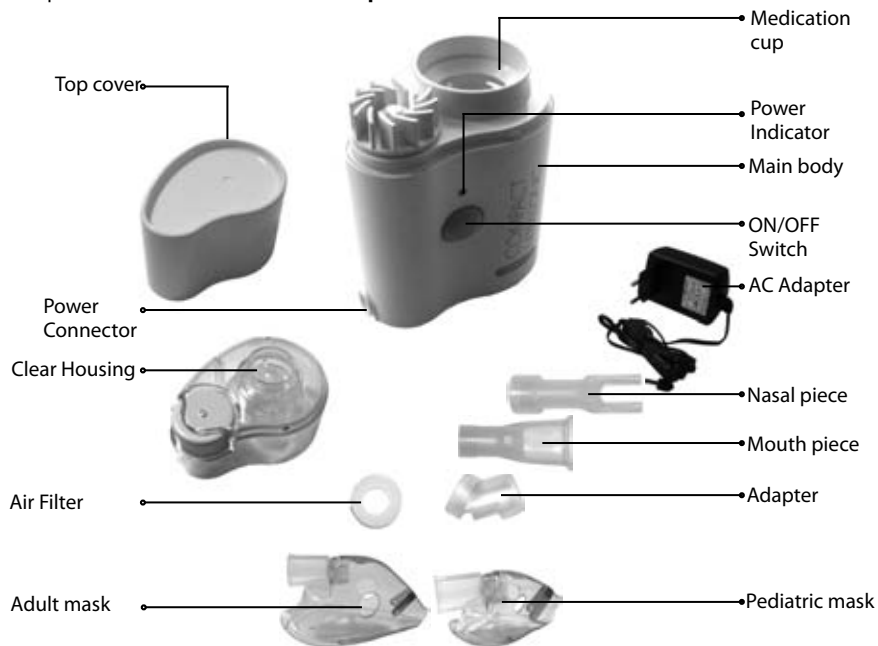RECOMMENDATIONS:

- Read the manual carefully before using the device and keep it in case you need further information in the future.
- Do not turn the device on if there is no drug solution into the medication cup.
- Disconnect the nebulizer when it is not being used.
- Do not remove the medication cup cover while the device is operating.
- Do not expose the nebulizer to a heat source.
- Do not pour more than 8 ml. of solution into the medication cup.
- Do not leave drug solution into the medication cup when the device is not being used.
- Do not scrape or pierce the medication cup cover.
- Do not touch the medication cup or the drug solution while the device is being used.
- Do not dip the nebulizer in water.
- Children may only use the device under adult supervision.
- To avoid infections and contagion the accessories should not be shared by different patients. They are intended for personal use.
- Use the unit intermittently. Maximum operating period: 30 minutes. Rest period: 30 minutes.
- This device can only be repaired by the local authorized service agents.
- Only use nebulize solutions prescribed by a doctor and follow his instructions carefully. Do not use balm substances

2. GETTING READY FOR USAGE

2.1. ASSEMBLING THE DEVICE

1. Take the unit and the accessories out of the box; carefully check the membrane placed at the bottom of the medication cup. If it is pierced or broken do not use the nebulizer and contact the authorized Service Centre.
2. Remove the clear housing and pour the medication into the medication container (8 ml. max.)
3. Put the clear housing.

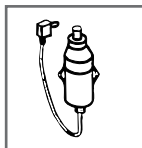
4. Insert the device to be used into it (nasal or mouth piece to be used with adapter).
5. Plug the unit and the power connector to the unit.

2.2. REPLACEMENT OF THE FILTER

1. The air filter has to be replaced if it turns dirty or torn.
2. Remove the air chamber from the medication cup cover and take out the filter with the help of forceps.
3. Place a new filter.
4. Put all the pieces back.

2.3.USING CAR LIGHTER ADAPTER (OPTIONAL)

Connect the adapter to the car lighter and the other wire end to the unit. Then operate the nebulizer as per the instructions previously described.

2.4.USING THE RECHARGEABLE BATTERY PACK (OPTIONAL)

Warning:

Precautions and Safety Measures

- Do not use the device under the sun, or close to a heat source.
- Never immerse the battery pack in water.
- Do not operate the unit for more than 20 minutes.
- It is normal for the battery pack to increase temperature while being operated.
- If the battery pack has not been used for a long period (3 months or more), repeat the initial charge.
- If the battery pack will not be used for a long period of time, it is advisable to store it fully charged.

Battery Charging / Initial operation

The Battery Pack is initially discharged and it is necessary to fully charge it before its first use to ensure the capacity and useful life of the battery.

1. Make sure that the ON/OFF button is OFF (0).
2. Connect the supply power cord of your nebulizer to the mains socket.
3. Connect the plug of the power supply cord into the opening of the pack base and charge 14 hours.

Battery Charging / Regular operation

1. Make sure that the ON/OFF button is OFF (0).
2. Connect the supply power cord of your nebulizer to the mains socket.
3. Connect the plug of the power supply cord into the opening of the pack base and charge between 6 and 12 hours, depending on the use. Do not leave the Pack charging for more than 24 hours.

Operating Instructions

1. Prepare the nebulizer as you normally do or follow the instructions above.
2. Connect the pack plug to the nebulizer.
3. Turn the ON/OFF button to the ON position (I) (the green light that indicates the state of the battery will illuminate). Turn the nebulizer on. If the red light is on disconnect the nebulizer plug and charge the battery pack.

Battery autonomy at full charge is approximately 30 minutes.

4. Once nebulizer use has been completed, turn the Battery Pack off and disconnect it.

Technical Specifications

Battery Type	nickel-metal hydride
Capacity	600mAH
Output	12V --- 1,1A
Input	12V --- 0,5A
Charge	time 14 hours Initial Charge 6 to 12 hours Normal Charge

3. CARRYING OUT NEBULIZING THERAPY

1. Press ON switch, the green light will turn on and the unit will start to operate. Adjust the air regulator to the desired intensity.
2. The unit will automatically turn off at 3 minutes. If you wish to continue nebulizing, check if the medication amount in the pool is the suitable one. Add more if necessary. Then press the ON/OFF switch twice and the unit will start to operate again.
3. Once use is over, unplug the equipment and disconnect the AC adapter. Remove the clear housing and dry the left medication using a clean cloth. There will always remain some solution in the basin which may not be used (between 0.5 and 1 ml).

4 CLEANING AND DISINFECTION

WARNING: it is highly recommended to clean the pieces before first use of the device and after each use

1. Wash all the pieces, except for the air filter, with warm water and detergent. Rinse with warm tap water to remove the detergent.
 2. Dip the pieces into a clean bowl containing 3 parts of warm water and 1 part of vinegar during 30 minutes, disinfectant can also be used.
 3. Wash your hands and take out the pieces from the disinfectant solution, rinse them with warm tap water and leave them on an absorbent piece of paper.
 4. Keep the pieces into a hermetic bag. Do not dry them with a towel.
 5. Clean the outer surface with a dry piece of cloth.
- Do not use any volatile fluid like alcohol or benzene.

5. POSSIBLE PROBLEMS AND HOW TO SOLVE THEM

If something wrong happens while the device is working, please check and correct the following points:

Symptom	Possible cause	Suggested solutions
Although the device is on, it does not work	1. Incorrectly plugged 2. No power 3. The AC Adapter is not connected	1. Check the plug is properly connected 2. Check the power is not gone out 3. Check the AC Adapter is properly connected
The mist does not come out	There is no solution into the cup	Pour some drug solution into the medication cup
Little mist is delivered	1. The air filter is dirty 2. There is no enough solution into the cup 3. Little power 4. Air excess	1. Change the filter 2. Pour some drug solution into the medication cup 3. Increase the power 4. Regulate the air flow
The device operates for a period and then turns off	Timer off	Press the key twice and turn it on again

Should the device not start working properly again, please contact the Service Centre; the address is stated in the Guarantee Card.

6. TECHNICAL SPECIFICATIONS

Method of nebulization	Ultrasonic Oscillation
Operating Voltage	220V, 50Hz
Class	□

Current	15W
Oscillation Frequency	2.5 mHz
Solution Output*	0.5 ml / min (minimum)
Particle Dimensions	1.5to5.7 μ m (MMAD 3.8 μ m)
Maximum load of basin	8 ml
Weight	260g
Dimensions	83.5 x 169 x 60 mm
Service Cycle	30 minutes on, 30 minutes off
Noise level (at 1 meter)	25 dB
Operating temperature and humidity	5 to 40°C , 30 to 95 %
Storage temperature and humidity	1 to 40°C, 10 to 95%

*This specification may vary with the kind of the medication being used and its viscosity. The specifications and the outer design are subject to change for improvement reasons without previous notice.

Symbols Key

	ON
	OFF
	Type B

Read the instructions

AC

Class II

7. WARRANTY

The ultrasonic nebulizer system device is covered by a 2-year warranty from date of purchase for any defects in the design or in the materials used. The warranty does not cover the accessories supplied with the device and those parts subject to normal wear and tear.

The warranty does not apply to damage caused by improper handling, accidents, not following the operating instructions or self-maintained alterations made to the device.

The manufacturer shall not be liable for any damages to people, property or pets, either directly or indirectly, caused by the non-compliance with the instruction book.

The warrantee is only valid upon presentation of the warrantee card which was correctly filled in and sealed.

8. WHAT IS NEBULIZING THERAPY

What is a nebulizer?

The name of the device comes from the Latin word nebula that means fog, cloud. The nebulizer system is a device that turns a liquid drug into a fine aerosol. Aerosol is a drug suspension. So, nebulizing therapy is an inhalation therapy with the help of aerosols.

Inhalation therapy has been used for treatment of lung diseases for a long time. The ancient civilizations of Egypt, India, China, the Middle East, Greece are known to have used inhalations of evaporations from menthol, eucalyptus, different plants.

For patients with lung diseases nebulizing therapy seems to be the most rational one since big drug doses come very fast to the respiratory tract directly.

How does the nebulizer system work?

Nebulizer directs an aerosol containing larger and smaller particles to the respiratory tract. The gravity makes the larger particles accumulate on the walls of the upper respiratory passages while the smaller particles have enough time to reach the lower bronchial branches. Therefore the drug comes to all parts of the patient's bronchial tree, including the smallest bronchi and alveole, and gets into the blood vessels on the walls of the alveole in big therapeutic doses with tiny or none side effects. Some drugs are very slow to be absorbed from the alimentary canal or are affected dramatically by the first passage through the liver. In such cases the way out is to use inhalation therapy. Nebulizing therapy can be used by patients of any age, it's suitable for small children and infirms.

What is nebulizing therapy used for?

Nebulizer systems are used by the patients with:

- acute respiratory diseases;
- obstructive pulmonary disease;
- chronic inflammatory processes in respiratory passages such as chronic bronchitis, bronchial asthma (incl. chronic or unstable ones, exacerbation of asthma), chronic pharyngitis;
- respiratory passages infections;
- abnormalities in secretion quality and secreting in bronchi;
- different types of coughing;

Nebulizing therapy helps to decrease the level of inflammation activity, to improve respiratory passage drain function, to lessen the bronchismus, to make the edema smaller and to protect the mucous membrane, etc.

What are the advantages of nebulizing therapy compared to other means of drug delivery?

Nebulizing therapy gives a possibility to quickly effect the inflammatory zone by big drug doses which increases the influence of the treatment and reduces the risk of side effects to other organs. Nebulizing therapy is the easiest way of inhalation treatment and it is also an economical means of treatments from the point of view of purposeful use of drug. Aerosol treatment with the help of nebulizing systems can be used for children, elderly and infirm patients.

Please, use only those drugs that were prescribed to you by your doctor, DO NOT execute self-treatment and DO NOT change the methods/frequency prescribed without consulting your doctor.

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